

Appendix 9: Replacing a corflo PEG Y adaptor

The end of the corflo PEG can split. The guidance below details how this can be replaced.



**Fig. 4 CORFLO* PEG: feeding connector assembly -
a: ENFit® Connectors; b: Skirt Stop; c: Tube Stop; d: Barb;
e: Skirt ; f: PEG Tube**

Close the fast release clamp on the PEG tube and disconnect the feeding set from the tube. Remove the old CORFLO* ENFit® feeding connector. If the PEG tube has split it may be required to trim the tube to ensure a tight fit of the new connector.

Untwist the threaded skirt from the new CORFLO* ENFit® feeding connector (Fig.4). This skirt is vital for the correct functioning of both the PEG tube and the CORFLO* ENFit® feeding connector

Put the PEG tube through the skirt. This skirt is vital for the correct functioning of both the PEG tube and the CORFLO* ENFit® feeding connector.

Insert the CORFLO* feeding connector into the tube, making sure the tube goes over the barb and reaches the tube stop at the bottom of the threaded portion.

Twist and push the skirt until the skirt stop is reached making sure it is a tight fit.